A.6000

A.6010

# Time•Markers

CE  
0123

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Radiopaque markers to  
study intestinal transit  
times.

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## TECNICAL SHEET

### Radiopaque markers for the study of intestinal transit times. For use with intestinal imaging.

Time-Markers are made up of 30 cylinders of a mix including white polyethylene (3.5 x 3.5 mm) with a specific weight of 1.3 gms. 12% barium sulphate is added to obtain a specific gravity higher than 1.

Markers are used to determine intestinal transit times in patients for whom a change of diet produced no improvement in symptoms of constipation, where diseases of expulsion may be suspected, or where a degree of colic constipation with slowing down of total transit times is manifest.

The main intended use of Time-Markers is to follow the precise path of food through the entire GI tract, and specifically in the bowel, whilst not influencing the performance of normal gastric and intestinal function.

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**Kutu adedi:** 20 Adet

### Sachets of 30 cylindrical markers:

- Administer an entire sachet of 30 cylinders with food (yoghurt, purée, soup).
- Perform 3 radiographic images on successive days.

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**Kutu adedi:** 20 Adet

### Sachets of 10 cylindrical markers:

- Administer a bag of 10 markers daily over 10 days
- Perform 1 radiographic on the 11th. day.

**Quantity per box:** 20x sachets